



BD BodyGuard™T Alarm Codes and Actions

Alerts, Alarms and Troubleshooting

Screen prompts	Result/cause	Possible actions
Occlusion Check Line & Syringe Press ► to Confirm	Alarm: Clamped line, occluded or kinked.	Release the clamp, flush/replace the access device or clear the occlusion.
Occlusion or Syringe Empty Check Line & Syringe Press ► to Confirm	Alarm: Clamped line, occluded or kinked, and the actuator has reached the minimum travel position.	
System Error, Press & Hold it+ for details If problem persists send PUMP for Service	Alarm: An internal system error has occurred. Two examples of system failure screen messages are shown here, refer to the pump service manual for a full list of error codes.	The user may be prompted to power off and restart, which may rectify the error. If the error recurs, take pump out of use. Press the @ key to obtain error message, record error code and summary of fault and return pump to designated service centre.
ERROR Startup MotMove Fail If problem persists		
Time & Date Incorrect date/time Press ► to restore	The internal backup battery has been depleted and the date/time values have been reset.	Press the key, then enter current date and time.
Program protection and Resume: <ul style="list-style-type: none"> Pre-Loading and syringe empty alarm clears a program from the pump memory. If the option is available to resume a program, a screen prompt displays. 		<div> 20 ml BD Plastipak Select ▲▼, Press ► </div> <p>If the syringe size/brand displayed matches the one used, press the key.</p> <div> Press ► to resume, ■ for New Syringe </div> <p>Press the key to retain the current program (ml/h rate is protected). Press the key to delete the current program which allows a new infusion to be programmed.</p>

Alerts, Alarms and Troubleshooting

Screen prompts	Result/cause	Possible actions
Keypad Locked	Only the , and @ keys are accessible.	Disengage keypad lock if further access required.
PUMP STOPPED Press ► to Resume	The infusion has been stopped.	Press the key to Resume the infusion or press the key to continue stopped state.
Program nearly Complete	Alert: Program is about to end / syringe is almost empty.	Prepare to change syringe or discontinue pump use.
Low Battery	Alert: Battery is almost depleted.	Prepare to change battery.
Pump Paused Too Long Confirm, Press ►	Alarm: The pump has been stopped / paused for more than two minutes without any key presses.	Press the key to resume the infusion, press the key to continue pause for another two minutes or power off.
End of Program Press ► to Confirm	Alarm: The program is completed and the VTBI volume is fully infused	Press the key to confirm, then change syringe or discontinue pump use.
End Battery	Alarm: Battery will fail imminently.	Change battery.
Syringe displaced, Check Syringe, Press ► to Confirm	Alarm: One or more of the syringe detection sensors is not detecting.	Check the syringe and re-seat as necessary. Check screen messages for assistance.